

M59 • 63

# Newborn Bathing and Nursery Care Model

M59 : Male  
M63 : Female

**⚠ Caution** | Do not leave this manual in contact with manikin skin. Ink marks will be indelible, due to pigment infiltration.



## Table of Contents

- Introduction  
Manufacturer's note .....P. 1
- Before you start  
DOs and DON'Ts .....P. 2  
Set includes .....P. 3





## ● Manufacturer's note

This training model is a life-size seamless manikin of a 0-4-week old newborn infant. Taking a bath, holding the baby in your arms, feeding him/her, changing diapers, dressing him/her in clothes; all these functions become very real with this life-like model. The manikin has the same weight as a newborn baby, and the anterior and posterior fontanel also simulate those of a real human infant.

The simulator is designed for training in bathing and nursing care. Do not use for improper purpose.

Please read the instruction carefully before use

## ● Features

1. Life-like, life-size newborn model
2. The neck of the manikin is not rigid, replicating a newborn's neck before learning to hold up his/her head.
3. The fonticulus anterior and posterior are palpable.
4. The limbs can bend and stretch, and the soft and supple material of the model allows trainees to practice baby care procedures just like those with real baby.
5. Waterproof: the model has a seamless, one-piece structure, in order to keep water from seeping inside.
6. The umbilical cord is removable.

## ● Available Practical Trainings

- Bathing
- Umbilical cord care
- Bottle-feeding
- Enema
- Ear and nose care
- Diaper changing, clothing
- Physical measurement
- Exercise and baby massage

## DOs and DON'Ts

### DOs

#### Handle with care

- The model is made of special composition of soft resin. Please handle it with the utmost care like an actual baby at all times.
- After each training session, dry the model with soft cloth and powder with talcum powder.
- Store the model with its clothe on, covered by the supplied cloth, away from heat, moisture and direct sunlight.

#### Bottle Feeding and Diaper changing

- Before feeding practice, put a diaper on the model. Feded fluid will be immediately discharged. .
- Perform the bottle feeding practice with water.

### DON'Ts

- Never wipe the model by thinner or organic solvent.
- Don't mark on the model with pen or leave any printed materials in contacted with their surface.
- Ink marks on the models will not be removable.

## Set includes

Before your first use, ensure that you have all components listed below.

### ● Set includes



Newborn model

Cloth



Cloth cover



Feeding bottle



Umbilical cord

### ● Material

Synthetic resin

### ● Size and weight

Size : 50 cm

Head circumference 33cm

Weight: 3 kg







Caution

Do not leave this manual in contact with manikin skin. Ink marks will be indelible, due to pigment infiltration.

▪ For inquiries and service, please contact your distributor or KYOTO KAGAKU CO.,LTD.

#### KYOTO KAGAKU CONTACT & ORDERING INFORMATION

---

 **KYOTO KAGAKU co.,LTD**

URL ● <http://www.kyotokagaku.com> e-mail ● [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)



#### ■ Head Office and Factory

Kyoto Kagaku Co. Ltd.  
15 Kitanekeoya-cho, Fushimi-ku  
Kyoto, 612-8388 Japan  
Telephone : +81-75-605-2510  
Facsimile : +81-75-605-2519

#### ■ LA Office ( for USA , CANADA and Mexico customers )

Kyoto Kagaku Co., Ltd. USA Office  
3109 Lomita Boulevard, Torrance, CA 90505-5108, USA  
Telephone : +1-310-325-8860  
Facsimile : +1-310-325-8867

---

The contents of the instruction manual are subject to change without prior notice.  
No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer.  
Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.  
Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback.  
Your cooperation is greatly appreciated.

2010.10.15