




USER GUIDE

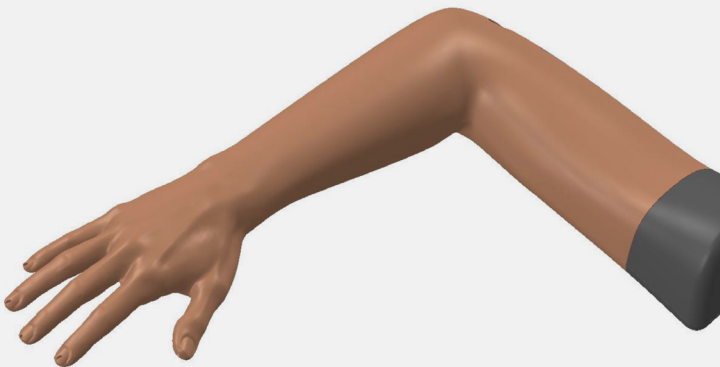

COLLES' FRACTURE REDUCTION TRAINER

70250

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED 2

SAFETY & CLEANING INSTRUCTIONS 3

SET UP 4


- QUICK SET UP 4
- REPLACING THE HAND SKIN 6

CLEANING & STORAGE 8


- CLEANING & STORAGE AFTER USE 8

PACKAGE SUPPLIED

COLLES' FRACTURE ARM
(includes 70251 Hand Skin)

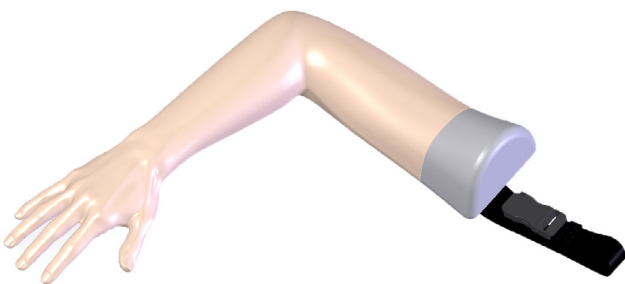

ADJUSTABLE HEIGHT STAND

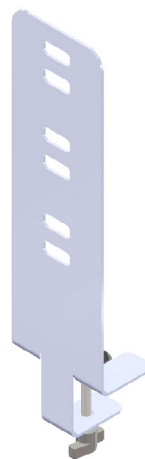

HEX KEY (x2)
Large (5mm) & Small (2.5mm)



STORAGE & TRANSPORTATION

Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR COLLES' FRACTURE REDUCTION TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

To avoid tearing when replacing the hand skin, you can apply talc to the inside, this will prevent friction occurring when fitting the hand skin.



If a product is damaged or worn, it should be repaired by Limbs & Things. To arrange this, please contact your point of purchase who will be able to provide a repair quote.

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.



Take care when removing cast so not to damage the skin

Tension must be set back to 0 (lowest tension) when product is not in use.



DON'Ts




Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.




Apart from hand skin removal, do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

Do not allow young children to use this product.

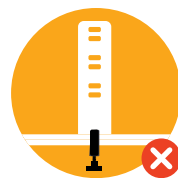


Stand not to be pulled on with force while training.

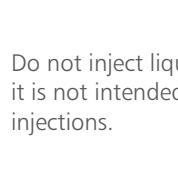


Do not inject liquid into the product, it is not intended for anaesthetic injections.

SET UP

The product is ready to use out of the box.

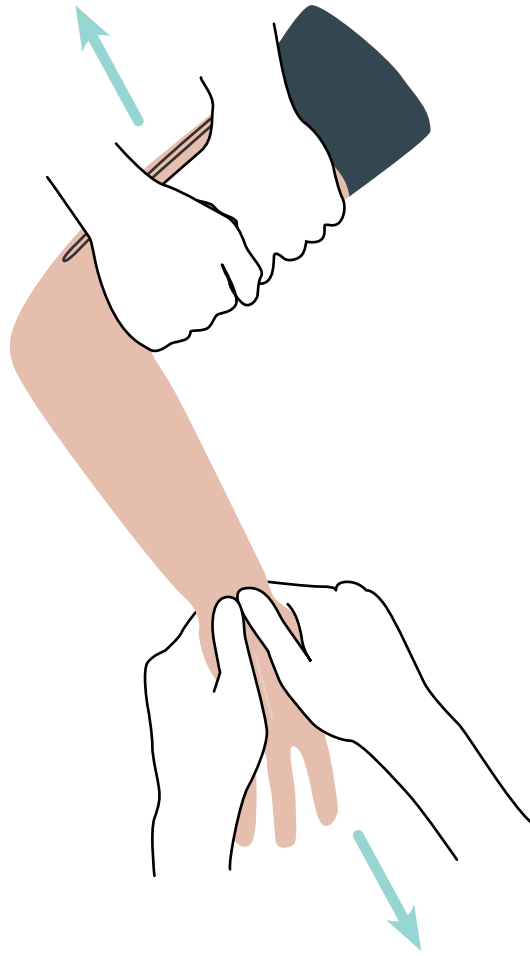


An **adjustable tension feature** allows for progressive levels of difficulty.

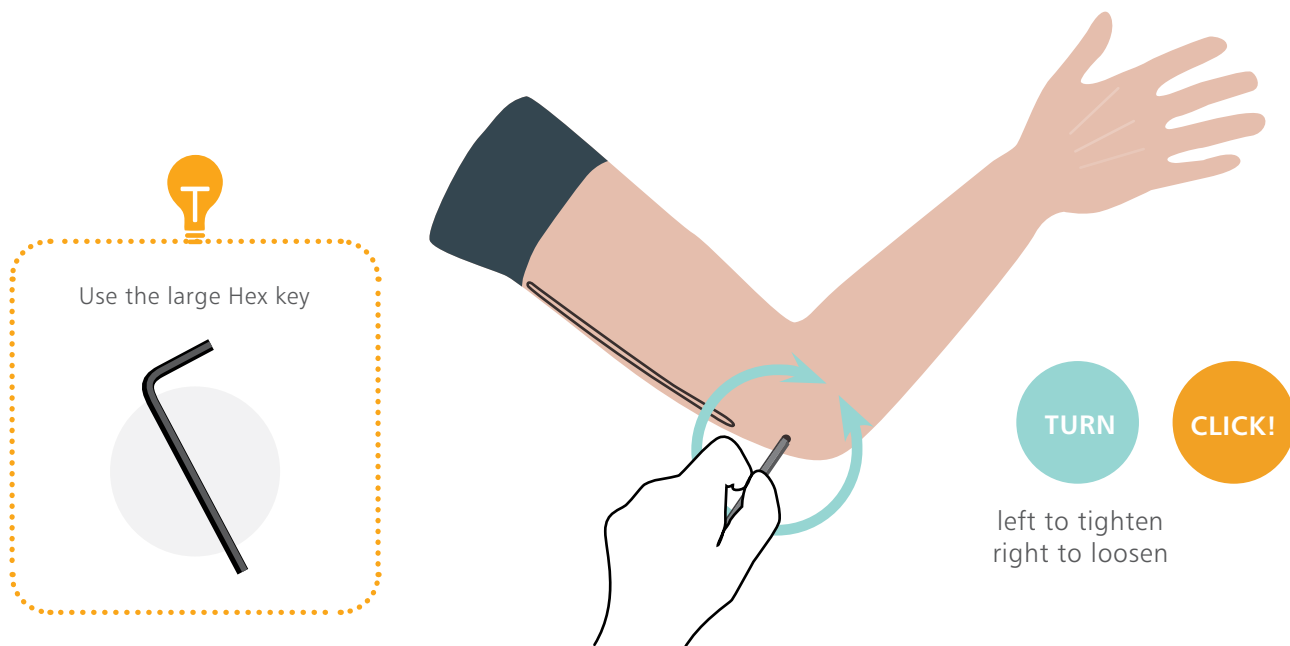


SET UP

The stand provides a **fixed reference point**.

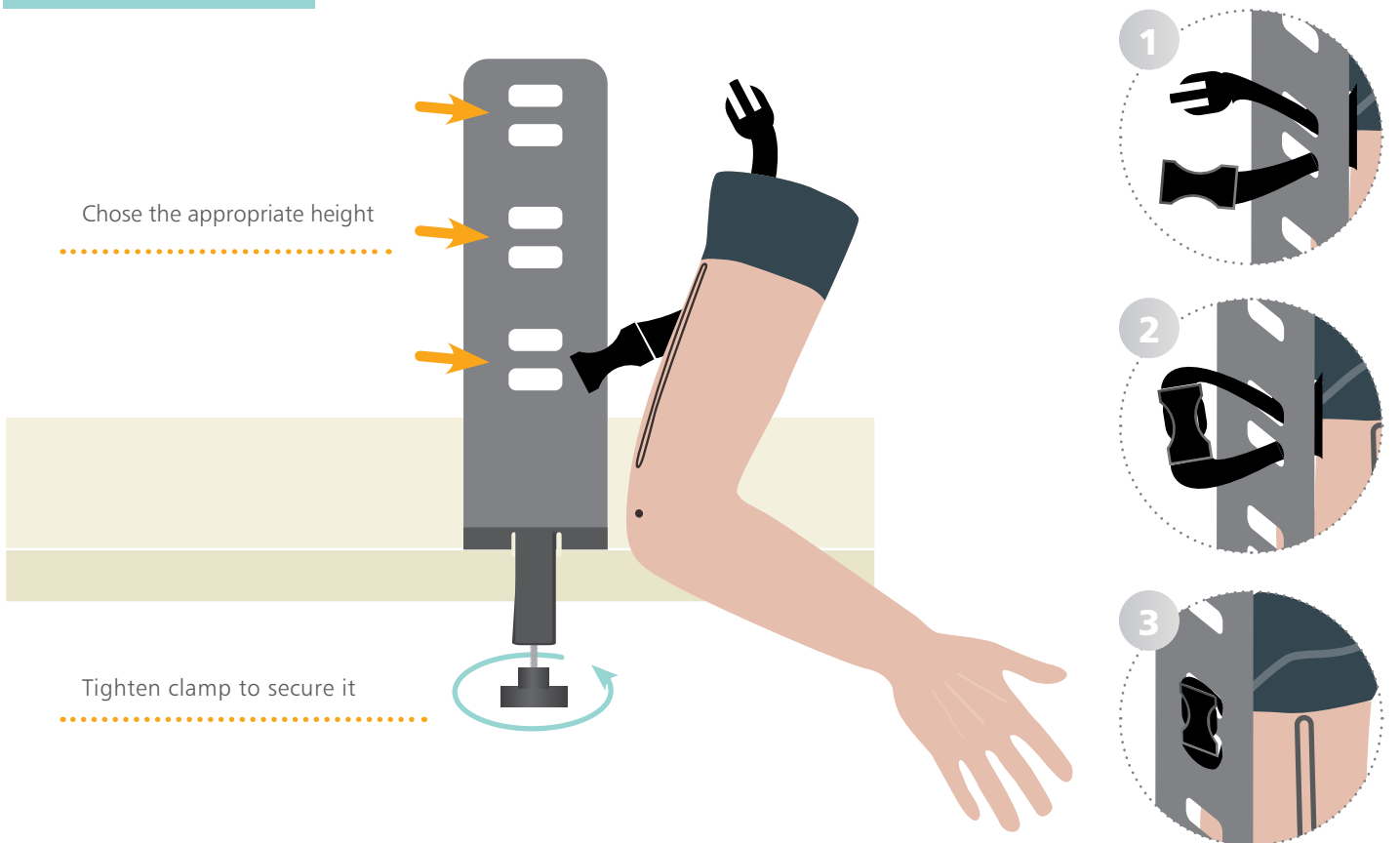

Product to always be used with a minimum of two people, conducting traction and counter traction, even when using the stand


Numerous **plastering techniques** can be performed.

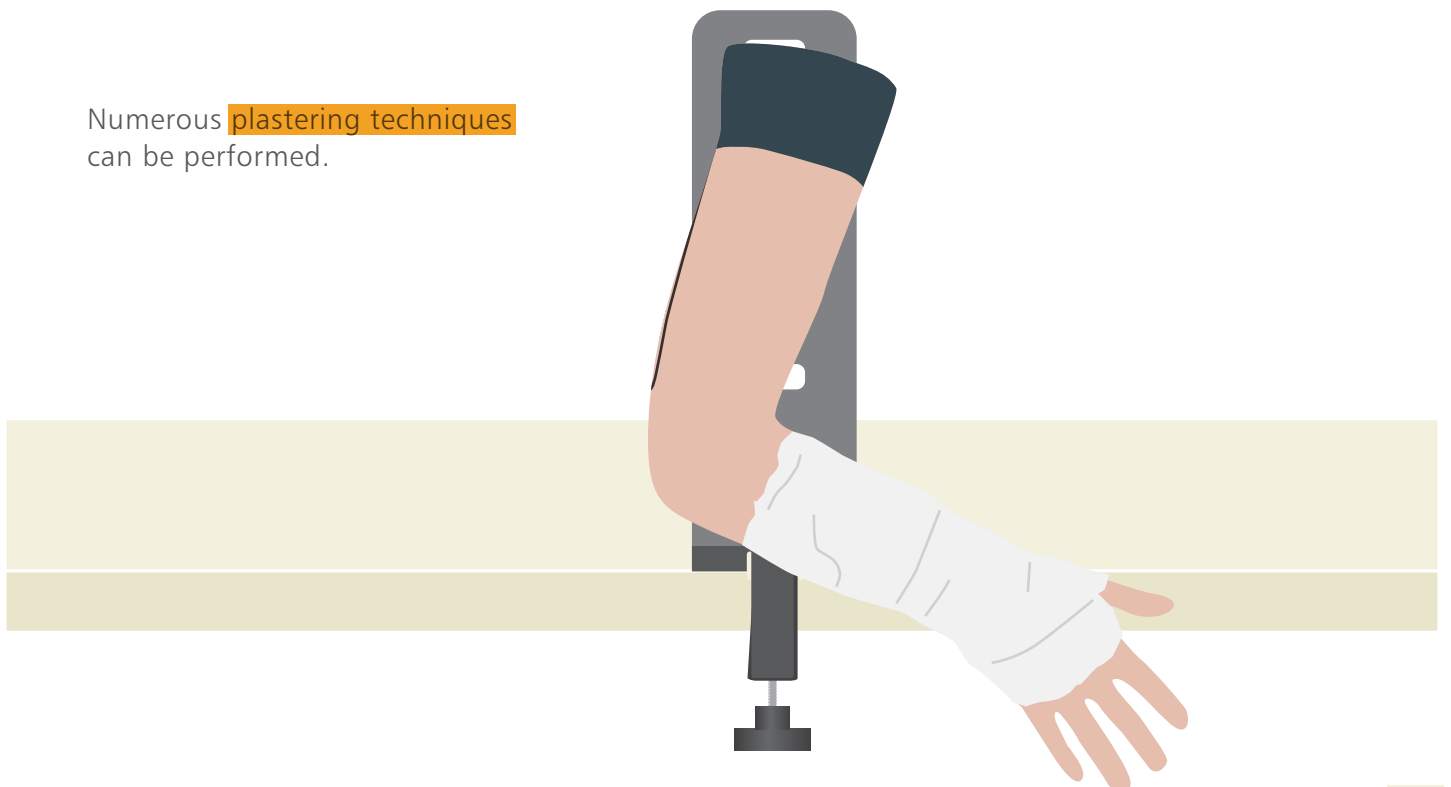




SET UP

REPLACING THE HAND SKIN

Remove the 3 screws.

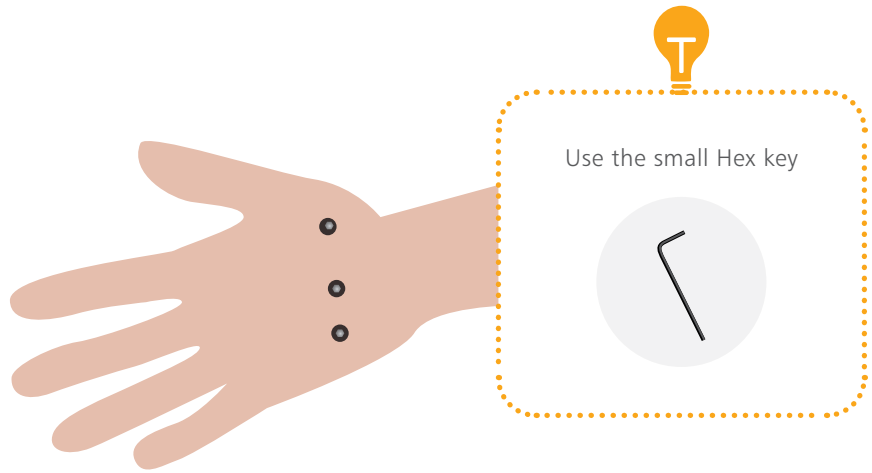


Remove the skin securing plate.

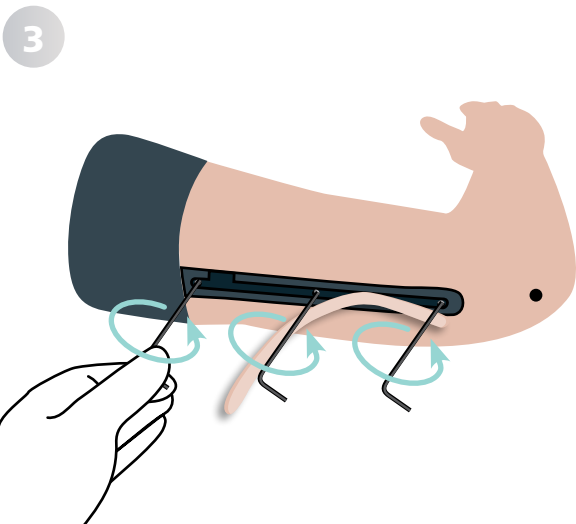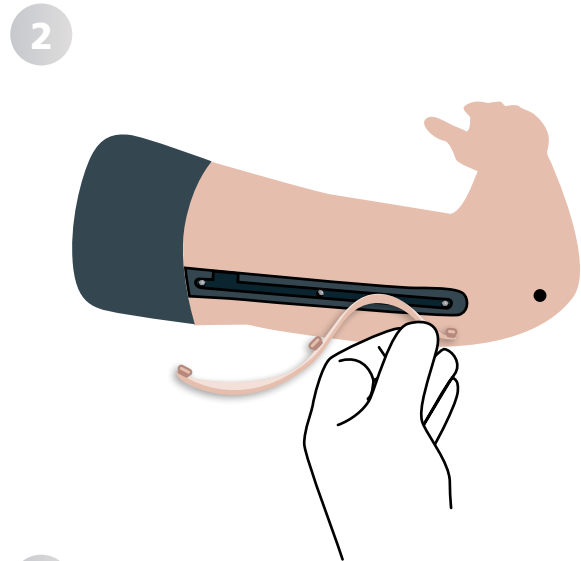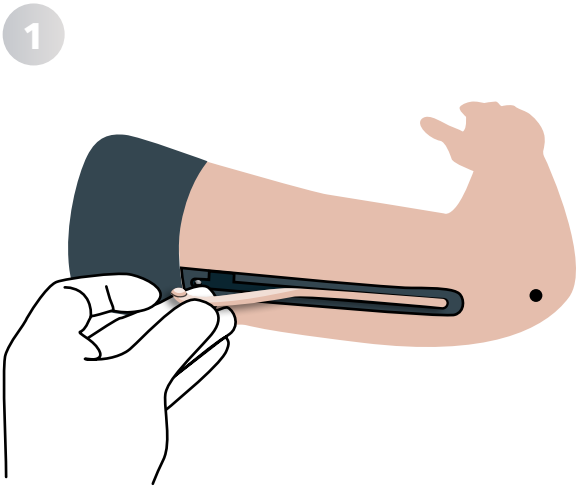


SET UP

REPLACING THE HAND SKIN

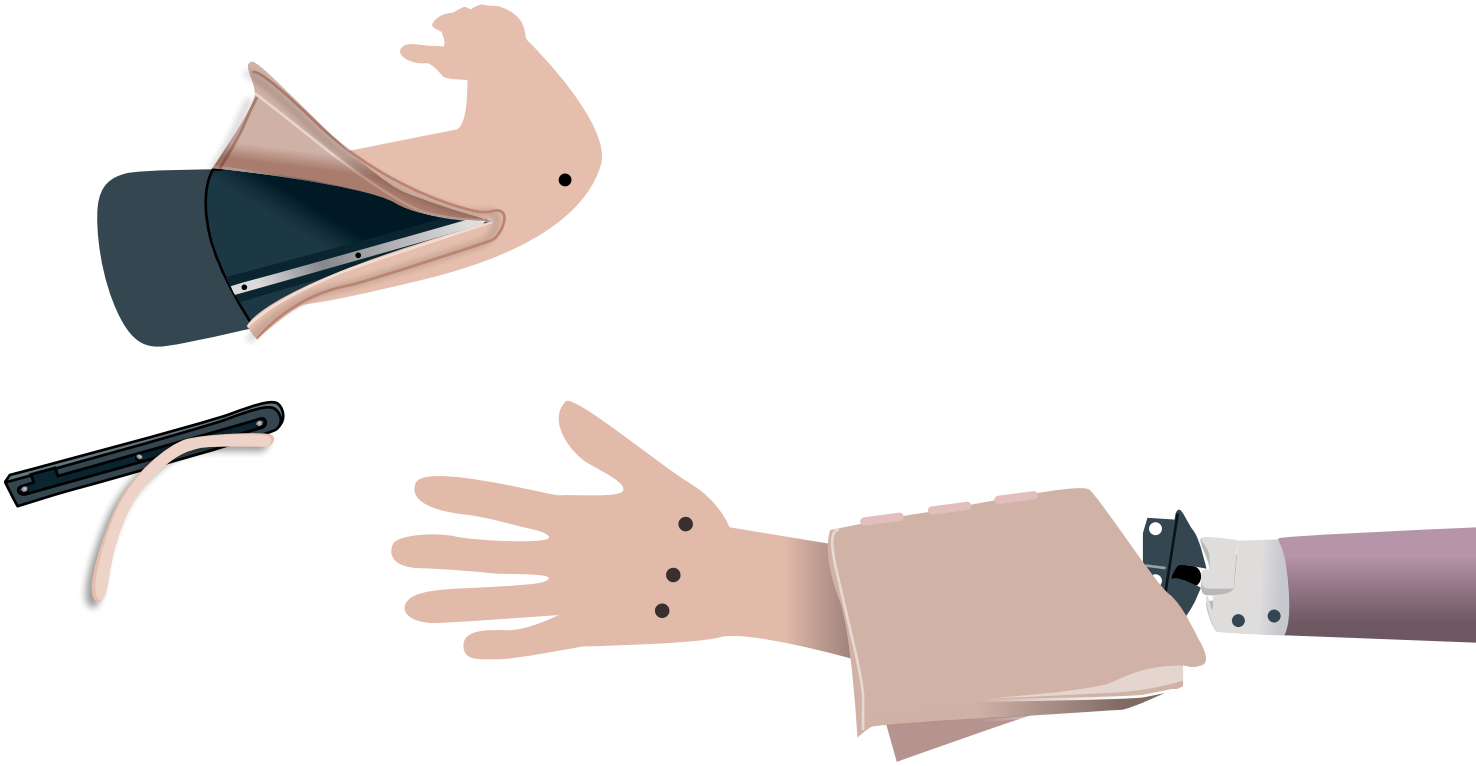


To replace the skin, reverse this process.
Take particular care when first inserting arm skin over the forearm.

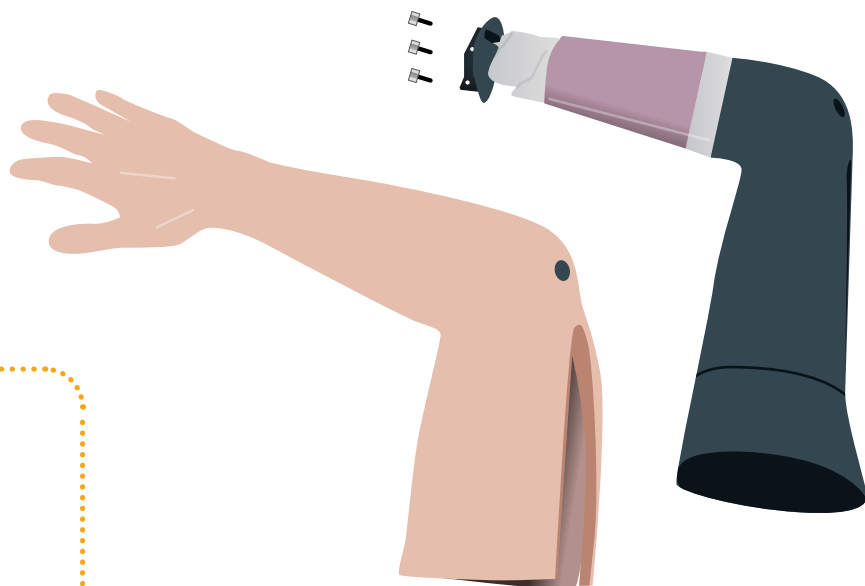

Apply talc on the inside of the
arm skin to make things easier

CLEANING & STORAGE

Clean off any surface dirt with a damp, soft cloth, using only warm water and mild detergent.

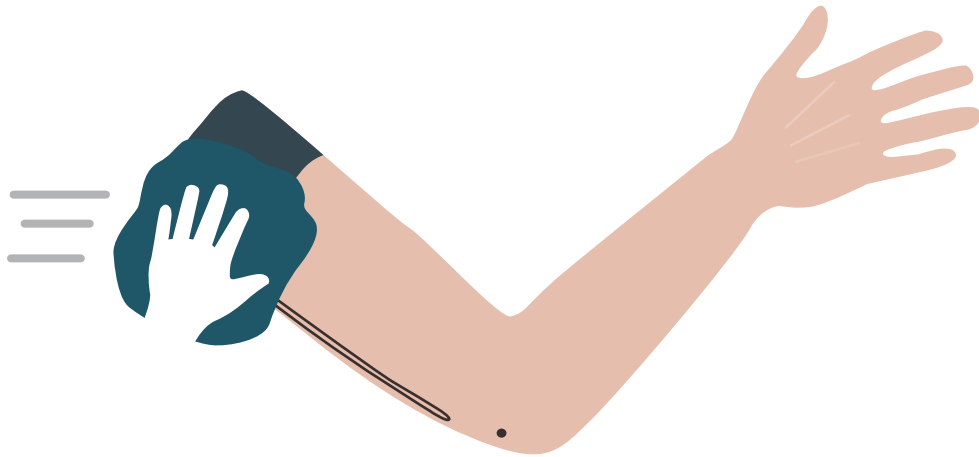

TAKE A LOOK
AT OUR **VIDEO**



VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings



Part No: 965-531

Issue 1, April 2021

© 2021 Limbs & Things