

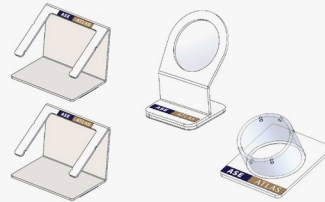


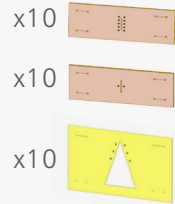


# ATLAS Upgrade Kit

## USER GUIDE

50401

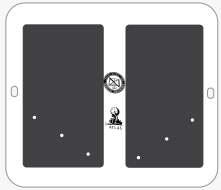


x3

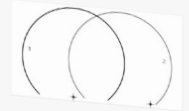


### TURN YOUR FLS BOX INTO ATLAS BOX

#### 1. Replace the lid

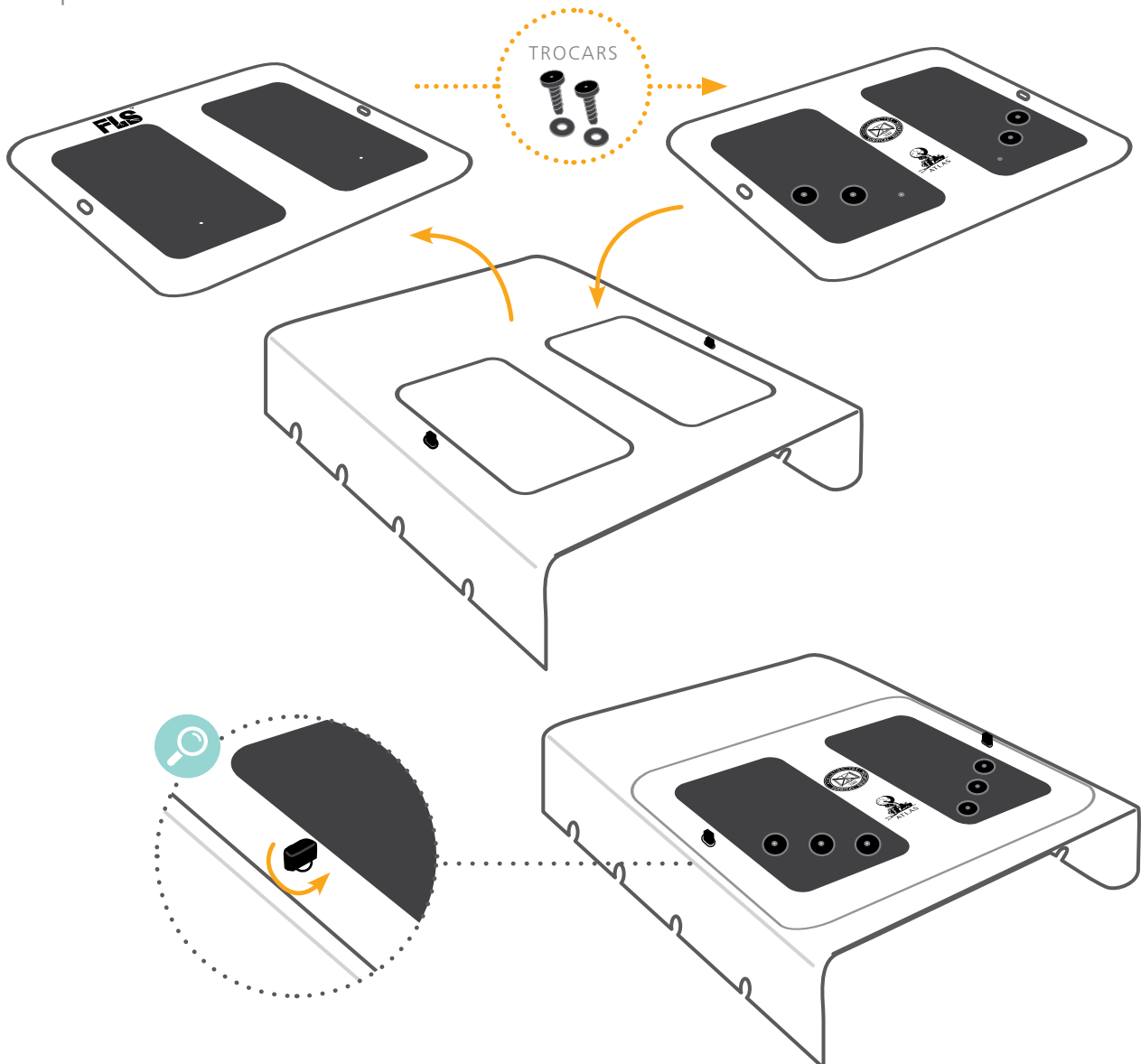

## 2. Place the sticker and velcro on the bottom of the box

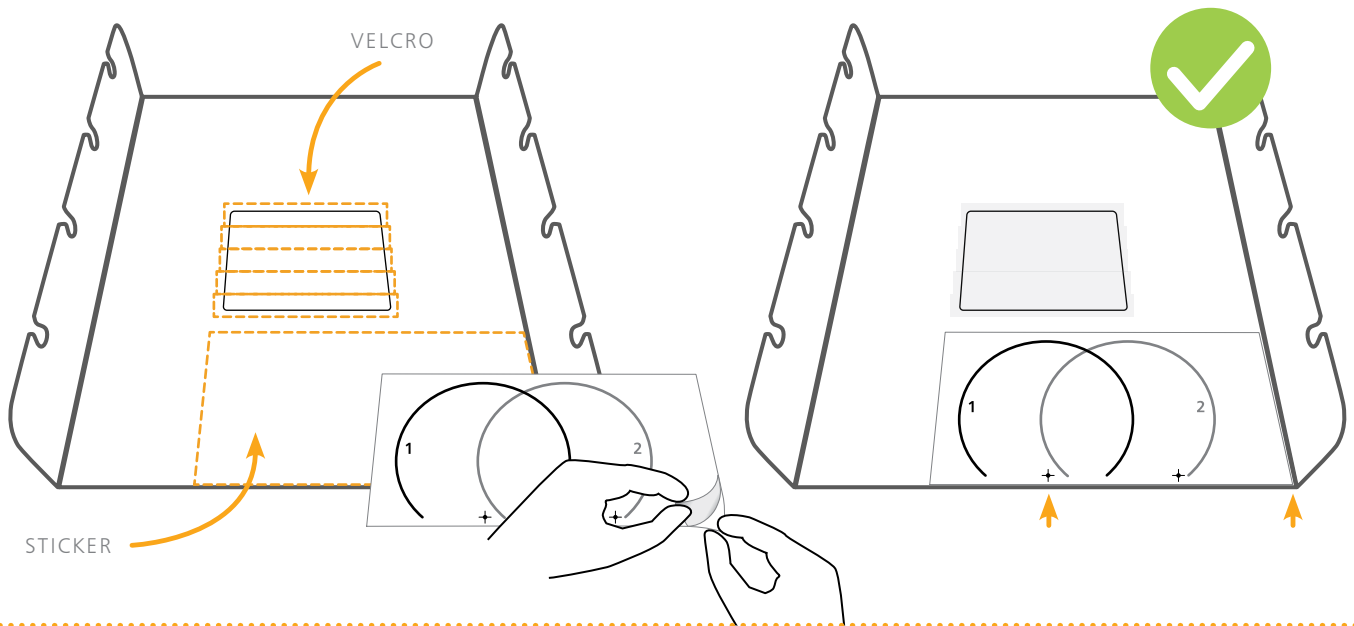

### ATTACHING THE STICKER

- Prepare the area using the 'wet method' (see below for further information)
- Carefully unpeel a small amount of the backing paper from the bottom right corner of the sticker and lay it down, aligning with the bottom right corner of the FLS base
- Make sure the bottom edge of the sticker is parallel with the front edge of the base
- Slowly pull the sticker backing towards the left-hand side of the base, holding it with your left hand
- Apply light sliding pressure to the sticker with your right hand to position it correctly as it lays down on the base
  - \* Be careful not to get the surface of the sticker wet as the ink may rub off with your hand


### USE THE WET METHOD

To make thing easier you can use a fine water mist to position the sticker:

- Fill a spray bottle with tap water
- After cleaning your surface area, hold the bottle about 12 inches away and spray 2 – 3 times, ensuring a light and even 'mist' is applied
- Apply the sticker while the area is still wet as this will keep it from adhering immediately and give you time to reposition if necessary
- Use a squeegee or plastic card to scrape the surface of the sticker running carefully to the edge to remove any air bubbles



## 3. Attach the clips to the inside of the top of the box

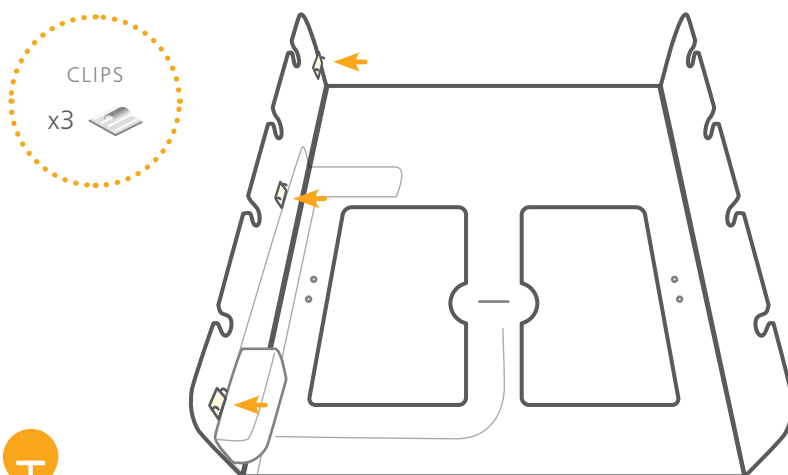

The purpose of the clips is to provide a method of routing the camera cable



### Clips position reference

