



USER GUIDE



ANAL SPHINCTER REPAIR TRAINER

This product is available in light and dark skin tones

61102

61122


FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• SECURING THE STAND	5
• FITTING/REMOVING ANAL SPHINCTER REPAIR BLOCK	6
CARE & MAINTENANCE	8

TAKE A LOOK
AT OUR **VIDEO**

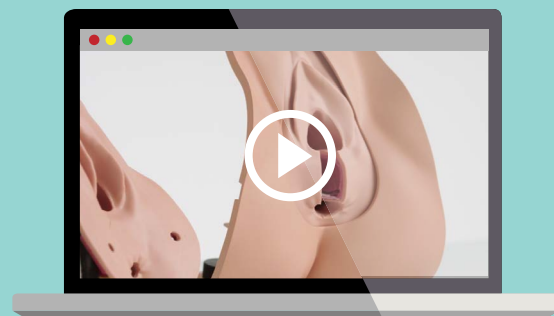

VISIT THE WEBSITE
limbsandthings.com


OR YOUTUBE CHANNEL
youtube.com/limbsandthings

PACKAGE SUPPLIED

Anal Sphincter Repair Block (x2)

61107

61127

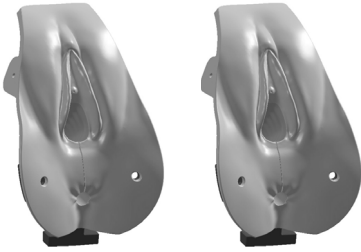

Perineal Repair Stand with Clamps

61109

61129



Perineal Repair Block Loading Handle

61110



Perineal Repair Illustration Pack

61112

61132



Lubricant

00293



YOU'LL NEED (NOT SUPPLIED)

Sutures



Forceps & Relevant Instruments



(We recommend using non-tooth forceps only to minimise damage to the repair block)

SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR ANAL SPHINCTER REPAIR TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.



Handle with care. The silicone is soft and can mark easily.

Use lubrication when performing a digital exam.



Remove sutures upon completing a repair. Great care must be taken to ensure that the block itself isn't damaged. We recommend using small sharp scissors to remove sutures.

Clean the product with a soft damp cloth or sponge, using only warm water with mild detergent, after every training session.




DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.

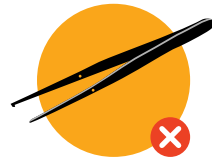

Do not use toothed dissectors/forceps when performing perineal repairs, to do so could cause damage to the silicone of the product. We recommend using non-toothed forceps instead.

Do not allow young children to use this product.



Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

STORAGE & TRANSPORTATION




When a training session is completed, and prior to packing away, remove all parts from the Stand component so that they don't experience undue tension for long periods of time whilst in storage.




Always ensure that the product is properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

1

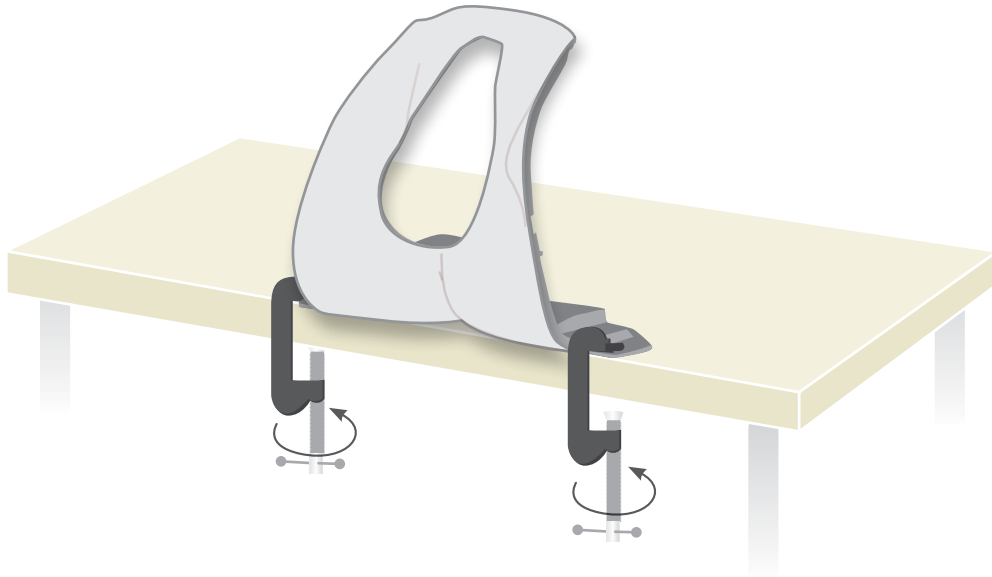


2

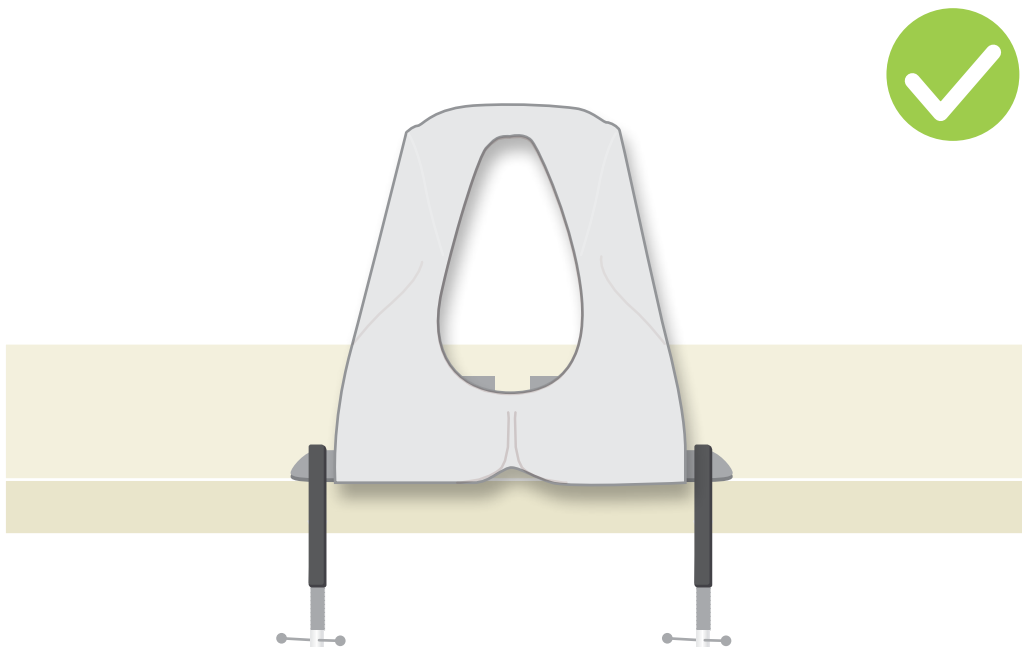

SET UP

FITTING ANAL SPHINCTER REPAIR BLOCK ON TO THE LOADING HANDLE


1

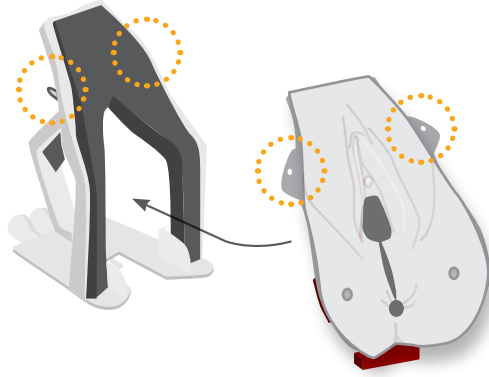

2









3

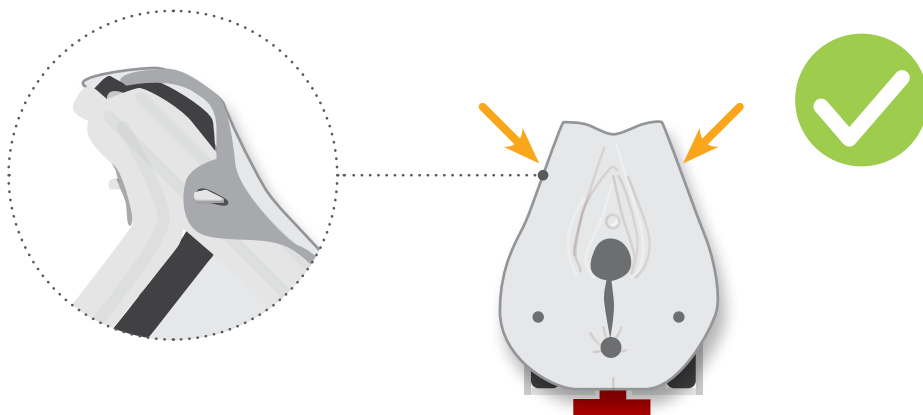

SET UP

FITTING TO THE STAND

click!

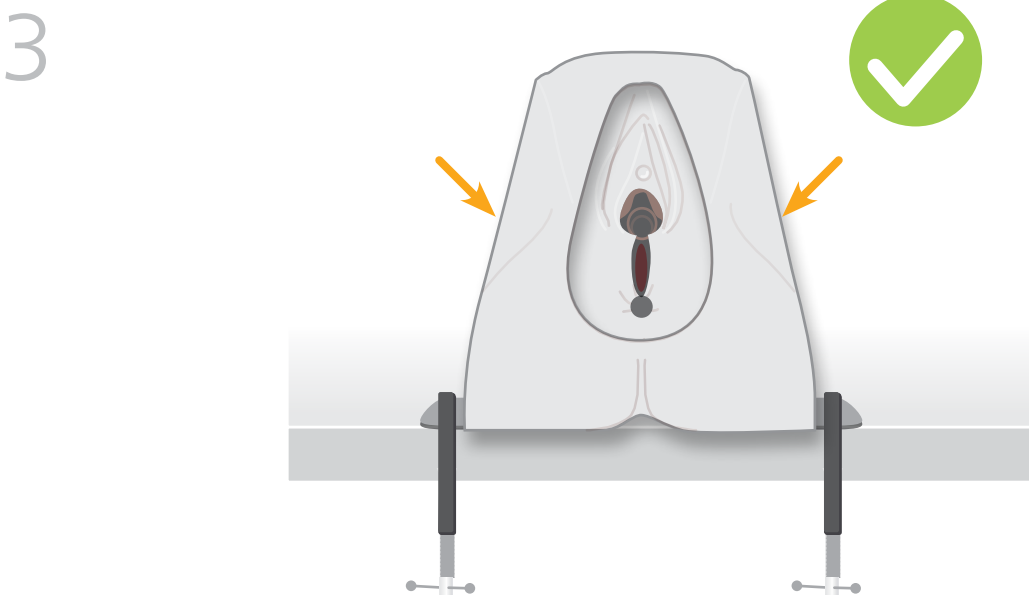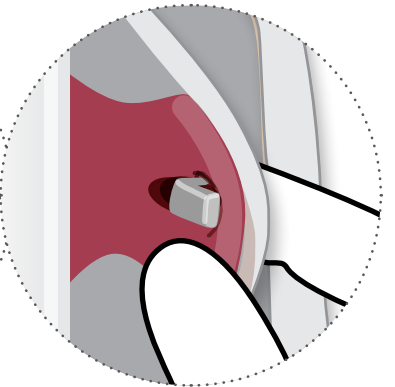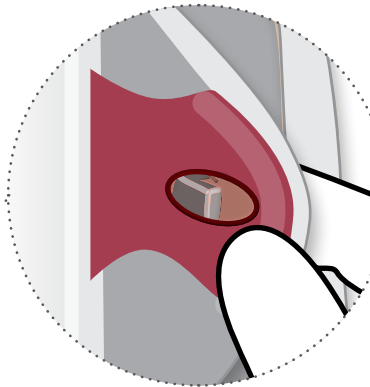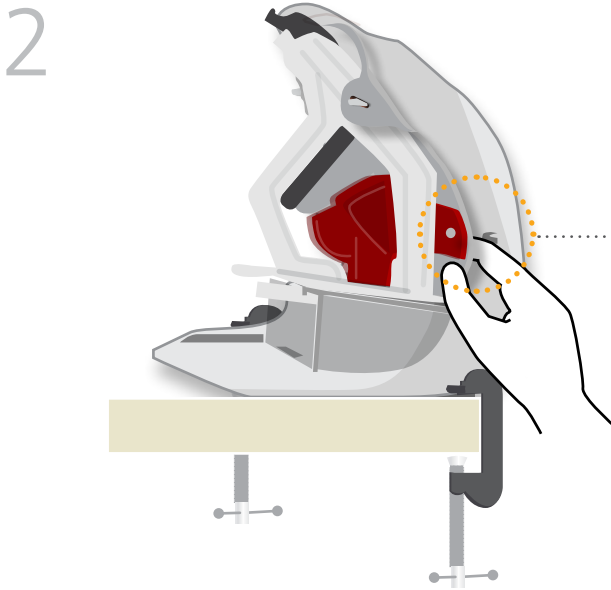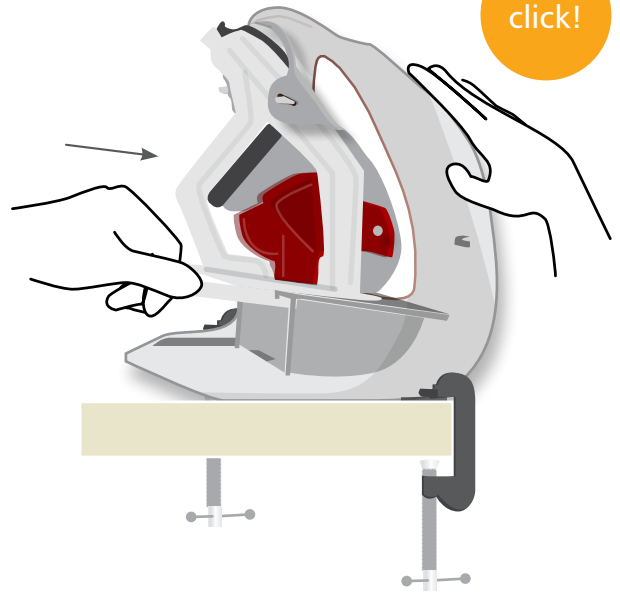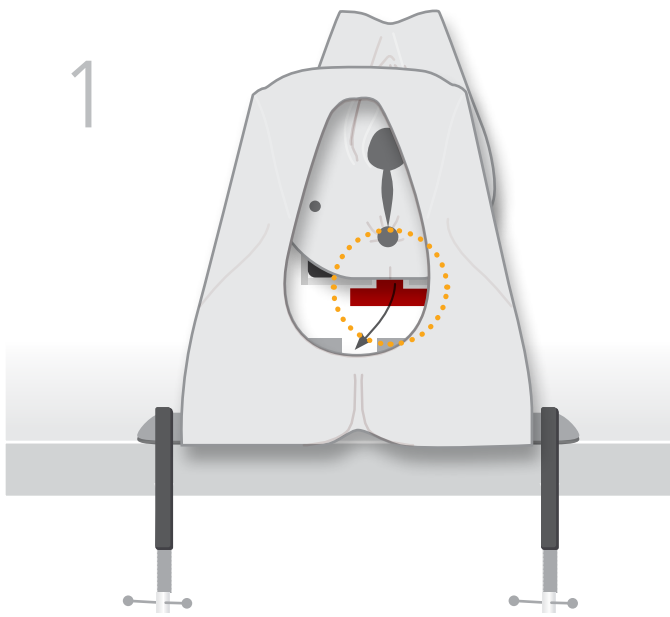

CARE & MAINTENANCE



Do not store it under tension



Remove sutures carefully



Clean with a damp cloth

YOU MAY ALSO BE INTERESTED IN...

EPISIOTOMY TRAINER

61119

61139

