



HAND & WRIST INJECTION TRAINER

Suitable for all levels of training, the Hand & Wrist Injection Trainer allows trainees and clinicians to practice and teach injections in five different locations. The model also highlights when incorrectly injected, ensuring trainees learn to avoid the median nerve.

The trainer comes equipped with an Android tablet, which works in unison with our newly developed app. The app, coupled with innovative sensory technology, provides learners with wireless, real-time feedback from the procedure.

70230

WITH TABLET

70232

WITHOUT TABLET





Learning has never been more lifelike

limbsandthings.com

ANATOMY

INJECTION SITES	BONE STRUCTURES	INTERNAL PRODUCT ANATOMY
<ul style="list-style-type: none"> - Carpal Tunnel - De Quervain's Sheath - Trigger finger - First Carpometacarpal Joint - Distal Radioulnar Joint - Trigger finger nodule - Median Nerve 	<ul style="list-style-type: none"> - Radial Styloid - Ulnar Styloid - Trapezium and Scaphoid - First Carpometacarpal Joint space - Distal Radioulnar Joint space - Lister's tubercle 	<ul style="list-style-type: none"> - Palmaris Longus - Flexor Carpi Radialis - Extensor Digitorum Group - Extensor Digiti Minimi - Abductor Pollicis Longus - Extensor Pollicis Brevis - Extensor Pollicis Longus

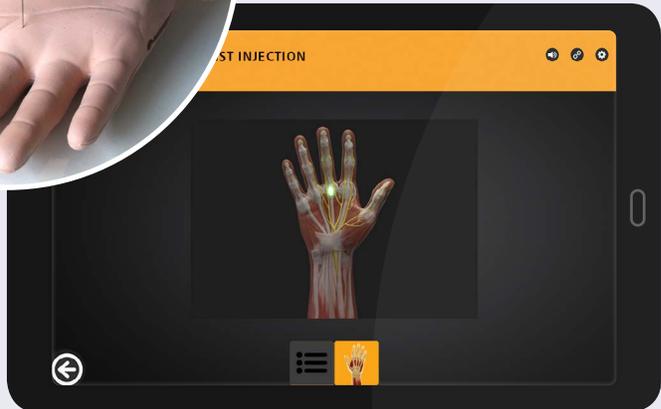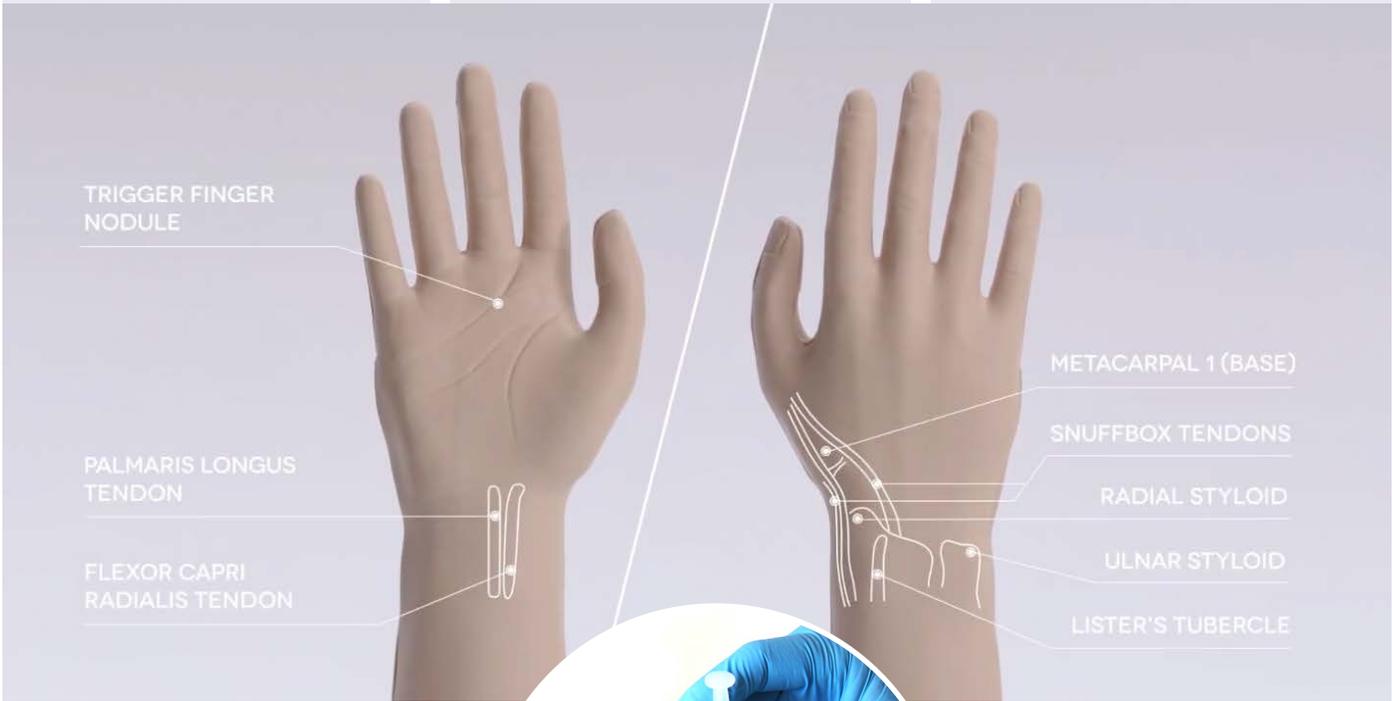



QUALITIES

REALISM

- Interactive model anatomy
- Anatomically correct
- Includes important ligament and skeletal structures

KEY FEATURES

- Facilitates injection in 5 key sites
- Presence of Median Nerve allows for negative feedback
- New app provides interactive real-time feedback
- Each site can withstand over 200 injections
- Skin can be easily removed and changed

VERSATILITY

- Lightweight and easily portable
- App can be set to English UK, English USA, French, Mandarin, Spanish, German and Russian

CLEANING & CARE

- Skin surface is washable using soap and water

SAFETY

- Fluid should not be injected into any of the sites, due to the electronics contained
- Latex free

SKILLS

- Model allows for injection into:
 - The Carpal Tunnel
 - De Quervain's Sheath
 - Trigger Finger
 - 1st Carpometacarpal Joint
 - Distal Radioulnar Joint
- Patient positioning and management
- Identification of anatomical landmarks

WHAT'S INCLUDED

- Hand & Wrist Model
- Hand & Wrist Skin (x2)
- Android Tablet
- Needle Set – 3 Green Needles, 1 Syringe
- Sharps Bin
- AA Batteries (x2)



CURRICULUM REFERENCES - UNITED STATES

UNDERGRADUATE

Recommendations for Clinical Skills Curricula for Undergraduate Medical Education, Association of American Medical Colleges, 2008, p.26

GRADUATE PAIN MEDICINE

ACGME Program Requirements for Graduate Medical Education in Pain Medicine (Anesthesiology, Neurology, or Physical Medicine and Rehabilitation), Accreditation council for Graduate Medical Education, 2017, p. 17

MUSCULOSKELETAL AND SPORTS MEDICINE

Musculoskeletal and Sports Medicine PM&R Curriculum Guidelines, American Academy of Medicine and Rehabilitation, p.5

GRADUATE PHYSICAL MEDICINE AND REHABILITATION

ACGME Program Requirements for Graduate Medical Education in Physical Medicine and Rehabilitation, Accreditation council for Graduate Medical Education, 2017, p.17

GRADUATE ORTHOPAEDIC SURGERY

The Orthopaedic Surgery Milestones Project, A Joint Initiative of The Accreditation Council for Graduate Medical Education and The American Board of Orthopaedic Surgery, 2015, p.35

RHEUMATOLOGY

Core Curriculum Outline, for Rheumatology Fellowship Programs, American College of Rheumatology, 2015, p.16 - p.85

MUSCULOSKELETAL RADIOLOGY

ACGME Program Requirements for Graduate Medical Education in Musculoskeletal Radiology, Accreditation council for Graduate Medical Education, 2017, p.2

NURSE PRACTITIONER AND PHYSICIAN ASSISTANT

Nurse Practitioner (NP) and Physician Assistant (PA) Rheumatology Curriculum Outline, American College of Rheumatology (ACR) and Association of Rheumatology Health Professionals (ARHP), 2017, p.8



REQUEST A QUOTE



REQUEST A DEMO



BUY ONLINE



CUSTOMER SUPPORT

Limbs & Things Inc.

P.O Box 15669, Savannah, GA 31416, USA
E: customerservice@limbsandthings.com
T: (912) 629-0357

limbsandthings.com